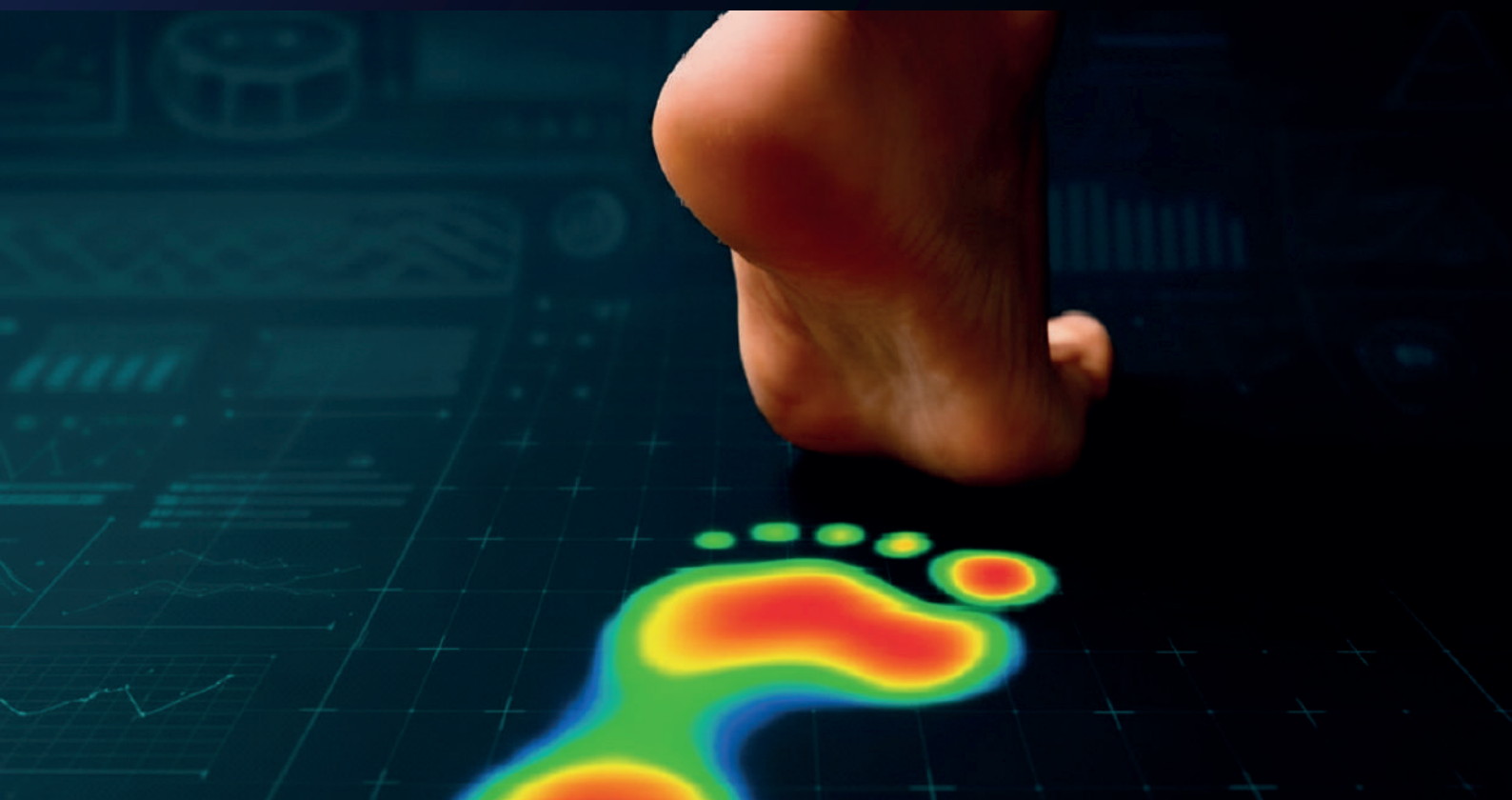
Fresco®

PODISENSE GO

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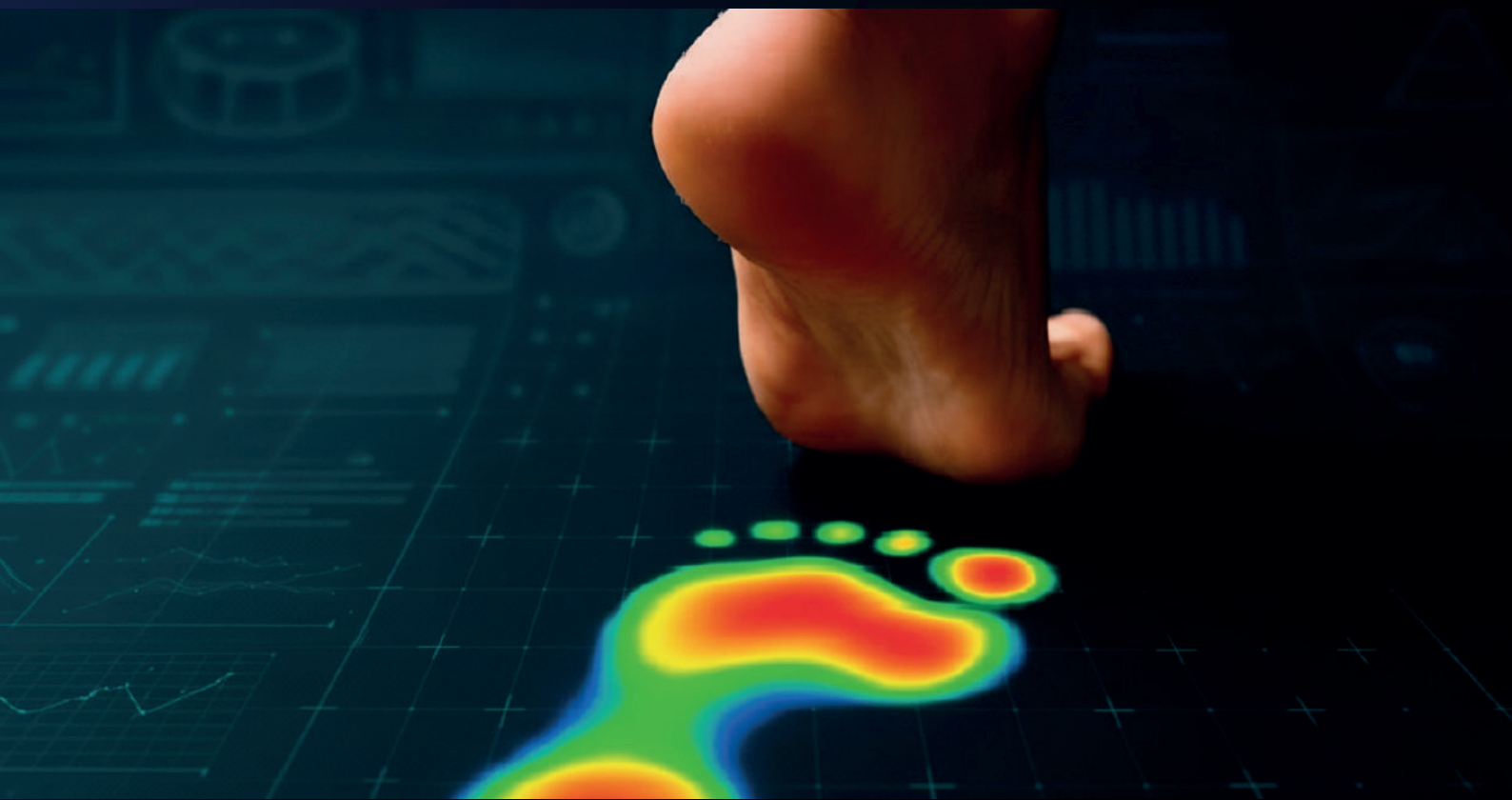
PODISENSE GO

PRODUCT MANUAL

A baropodometric platform that helps clinical professionals simplify foot plantar analysis

1. SYSTEM ELEMENTS

ULTRALIGHT

The lightest & only rollable
Baropodometric Platform on
the Market

0.5 kg / 3 mm

Compliant

SOFTWARE SUITE

The Only Cloud Solution
on the Market

Security

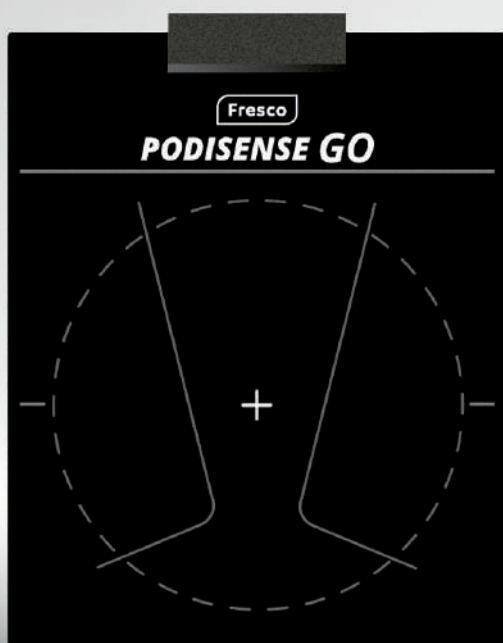
Scalability

Online Support
Software Updates


Predictive

2. COMPONENTS OF PODISENSE GO

1

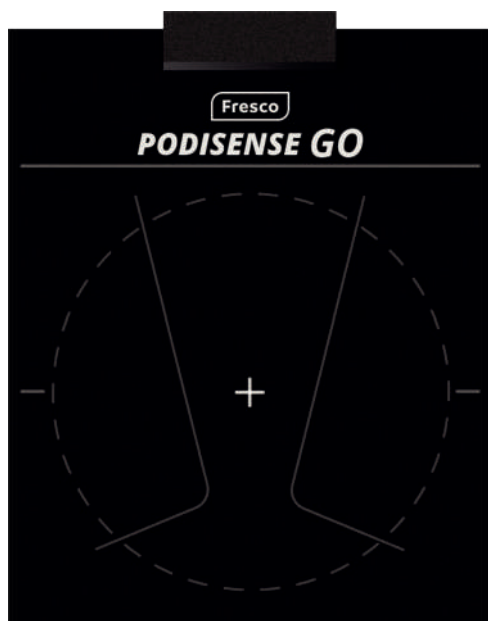Podisense GO
Plataform

2

Transportation Bag

3

Podisense
"HUB"

4

Power Cable

5

LAN & Micro USB Cable

CE CERTIFICATE EU DECLARATION OF CONFORMITY

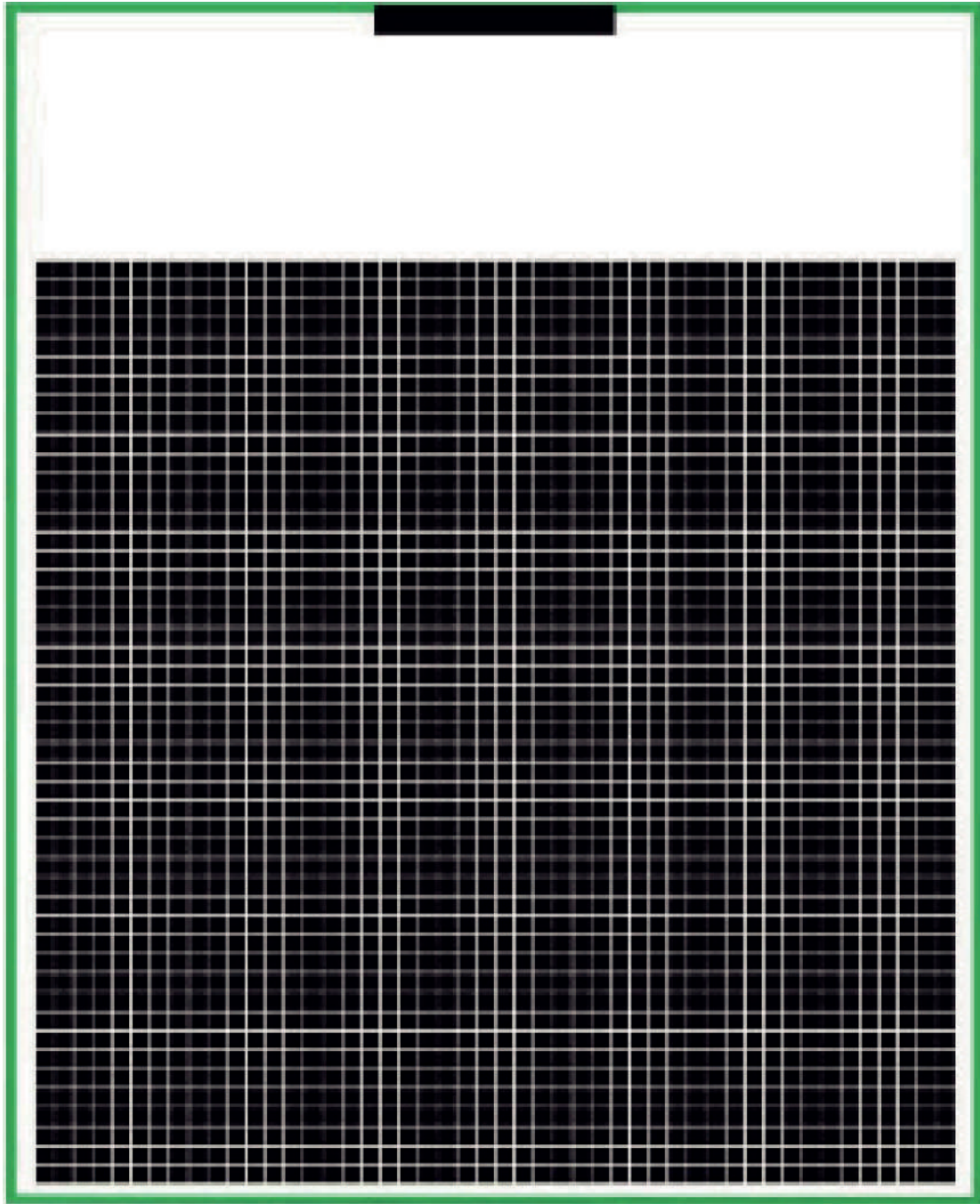
Keep the device away from water , fire , humidity or hot environments
Do not attempt to disassemble , repair , or modify the device
Do not use damaged charger or USB cable to charge the device
Do not use any other chargers than those recommended

3. ACTIVE SURFACE

Matrix 48 x 48 elements

Distance between centers 10 mm

Effective area W480 mm x L480 mm

4. PODISENSE GO MAIN FEATURES

PLATAFORM FEATURES

Overall Area (L x W) [mm]	624 x 510
Sensing Area (L x W) [mm]	480 x 480
Number of Sensor Elements	2304
Matrix Topology [R x C]	48 x 48
Sensor Element Resolution [mm]	10
Sensor Element Shape	Squared
Sensor Element Size [mm]	6

PRESSURE SENSING FEATURES

Relative Pressure Accuracy	10%*
Pressure Range	20-10,000 mm Hg

HARDWARE FEATURES

ADC Resolution	8-12 bits
Scanning frequency [Hz] Real	Up to 100**
OS & Platforms Compatibility	Sensing HUB
Standard Connectivity Hardware	USB serial
Standard Connectivity Enabled	USB serial (24/7)
Connector Power	USB power cable
Built in Battery	No

* This device has been developed to deliver autocalibration based on weight patient

** Varies depending on different settings

5. SENSING HUB MAIN FEATURES

EXTERNAL

External dimensions 93 x 63 x 25mm

Body Material Plastic PLA

Sample image of the hub
Final appearance subject to change

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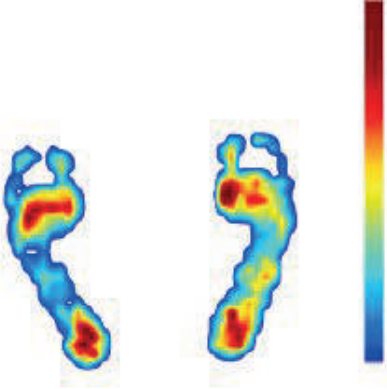
7. INSTALL THE PLATAFORM

- 7.1. Open the transportation bag
- 7.2. Remove the Plataform from the transportation bag
- 7.3. Remove the velcro strip from the Plataform
- 7.4. Place the packaging material (foam and velcro strip) into the transportation bag.

Keep the transportation materials in a safe place to use it to move the device from one place to the other.
Please, package the plataform in the same way as received.

8. CONNECT THE MAT WITH THE PODISENSE HUB

- 8.1 Connect the micro USB to the port at the electronic module

8.2 Connect the USB A to the Podisense Hub Please use the upper blue USB port as in the attached picture

8.3 Plug the Podisense Hub to the main to power it and wait 2 minutes to allow the system to start

Please, read the next section carefully to setting up the WiFi configuration

9. INTERNET CONNECTION SETUP

FIRST TIME CONFIGURATION

When you receive your Podisense Hub, you will be able to connect it via Ethernet cable connection or Wireless Connection

9.1 ETHERNET CONNECTION

- Connect the ethernet cable in the RJ 45 socket provided in the Podisense Hub
- Your device is ready to work Follow the instructions of the section 10

9.2 WIFI SET UP FOR THE FIRST TIME

Scan the QR code of the Access point provided

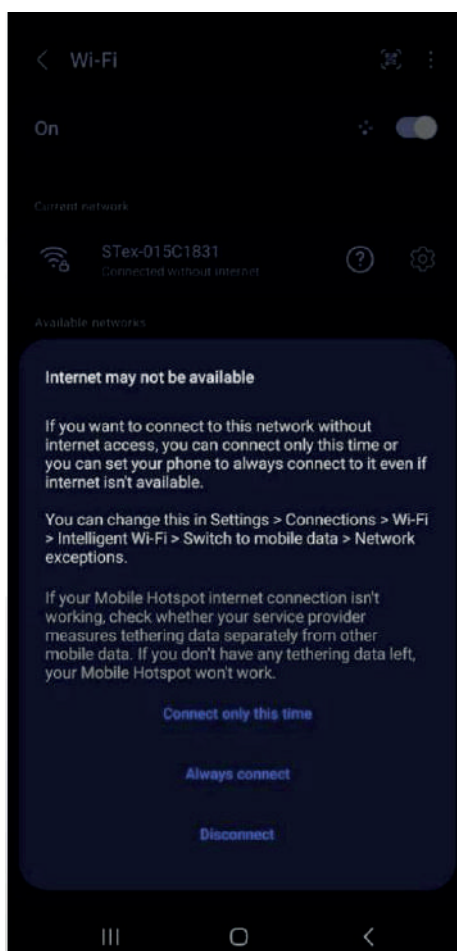
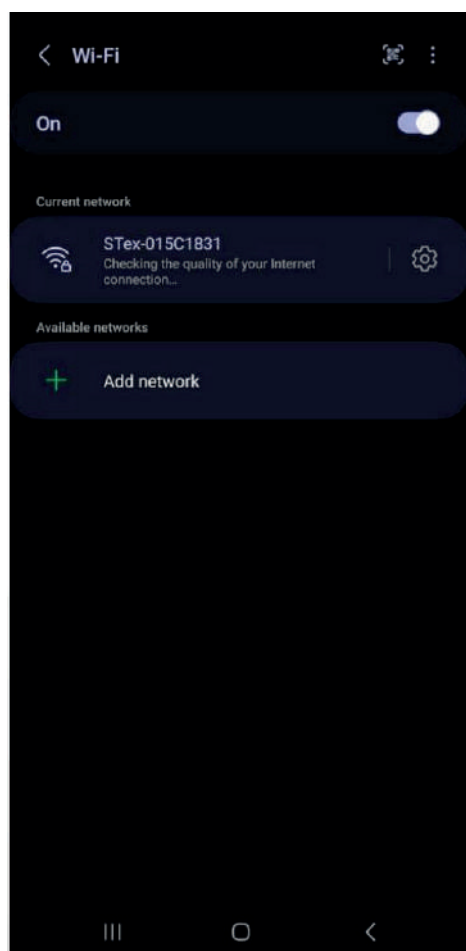
Please be sure that you are using the application "Camera" of your phone to read the QR code

The QR shown is an example. Use the QR you have received with your Podisense Hub

Your device will be connected to the Access Point of the Podisense Hub

When the warning appears informing that the selected network does not have Wi Fi connectivity; select the option "Connect only this time"

If you have troubles to set up the Wi Fi configuration, please can see how to configure it manually at the FAQ's section

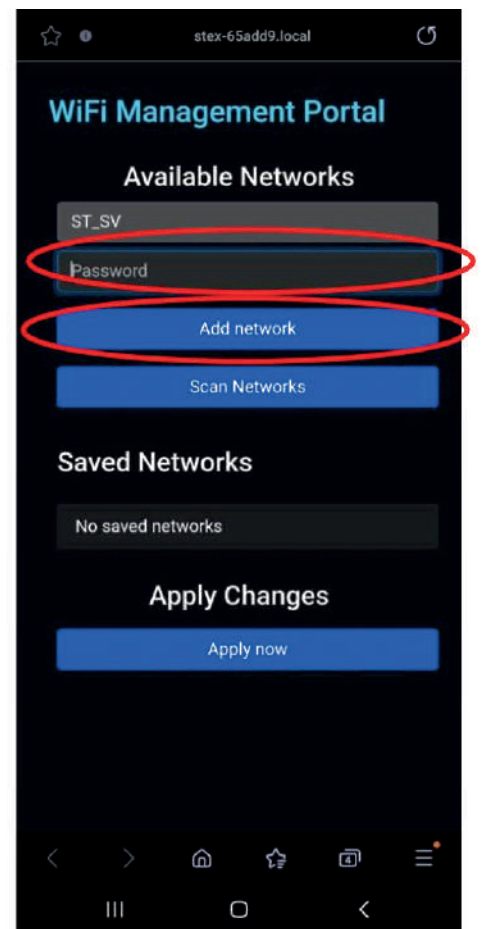
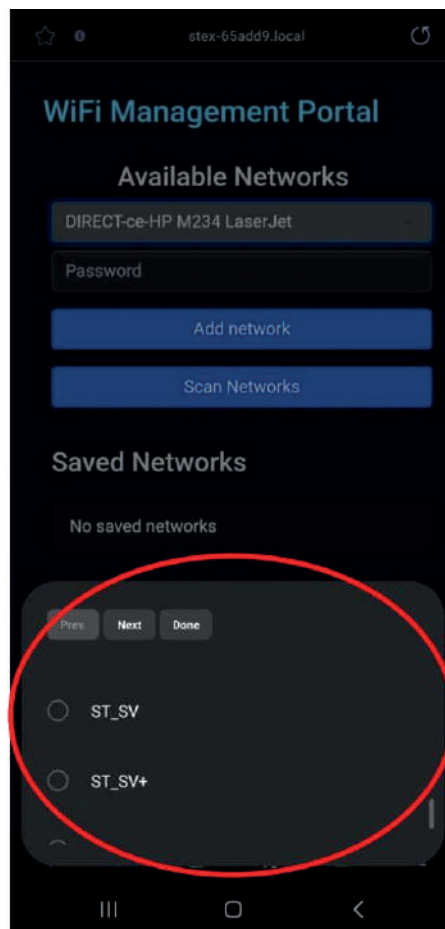
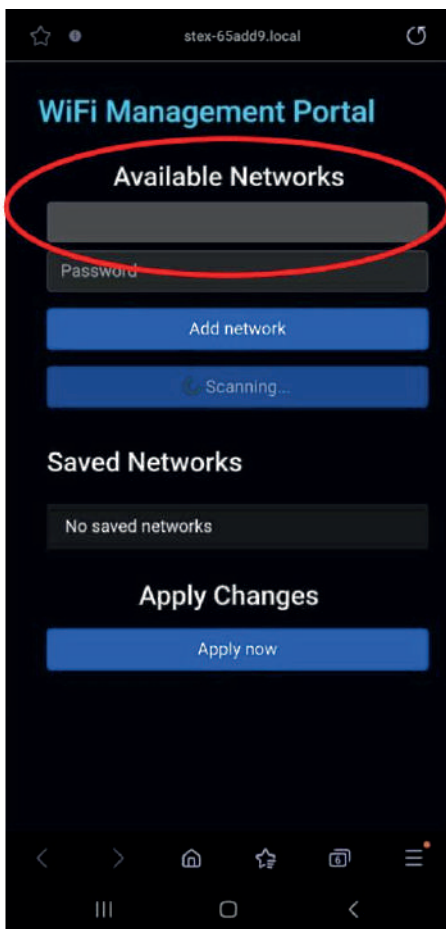
Scan the QR code “Portal (hostname)” to get access to the WiFi Management Portal

The QR shown is an example. Use the QR you have received with your Sensing Hub

From the WiFi Management Portal, search and select the name of your WiFi Network and fill the password


Click on “Add network”

You Will get a message confirming that the network is saved

Click on “Apply now” to save the network

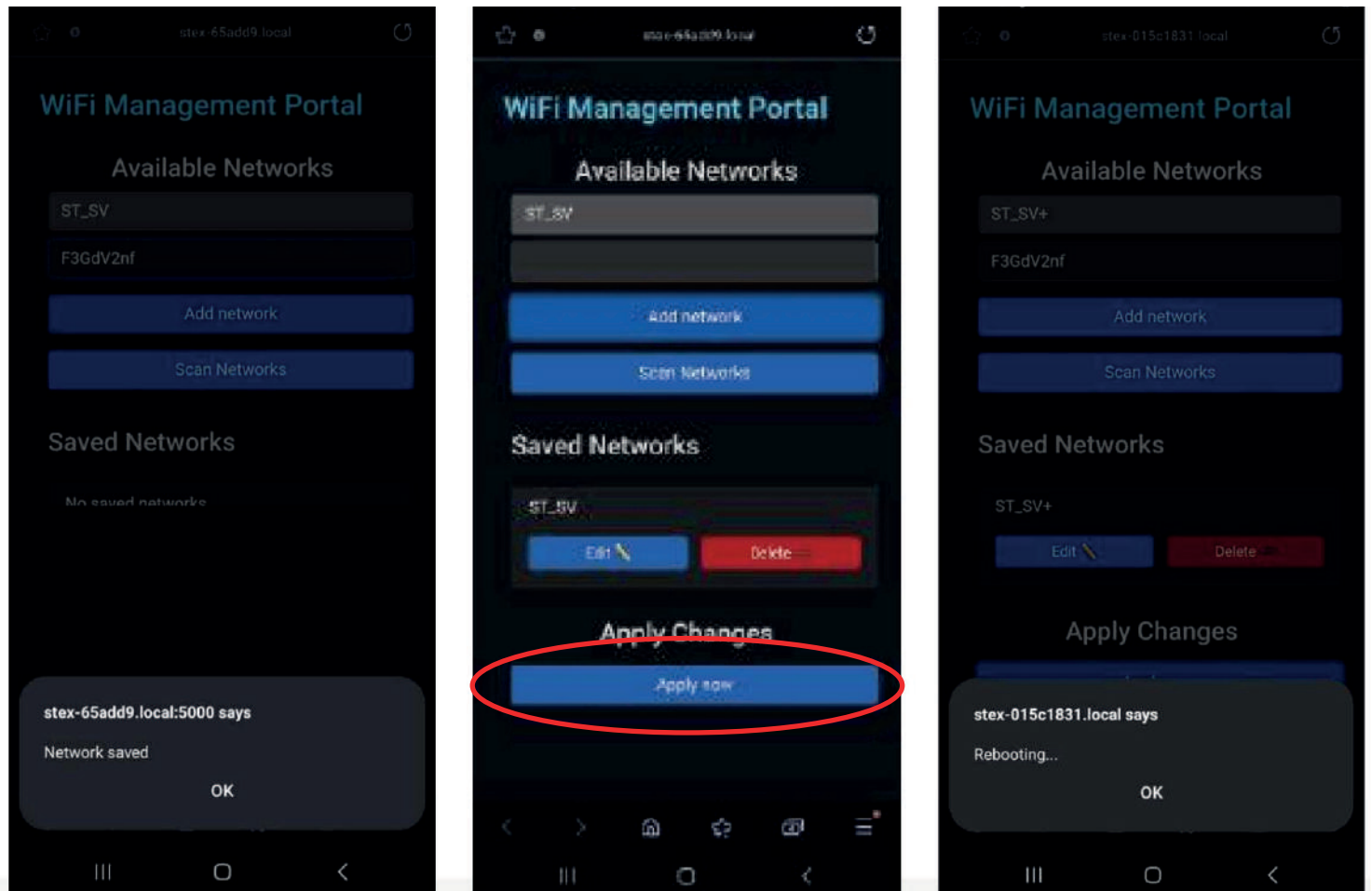You Will get a message confirming that the Podisense Hub is rebooting.
Wait 3 minutes for the rebooting process and to continue

The WiFi configuration is completed. You can use the system following the instructions of the section 10

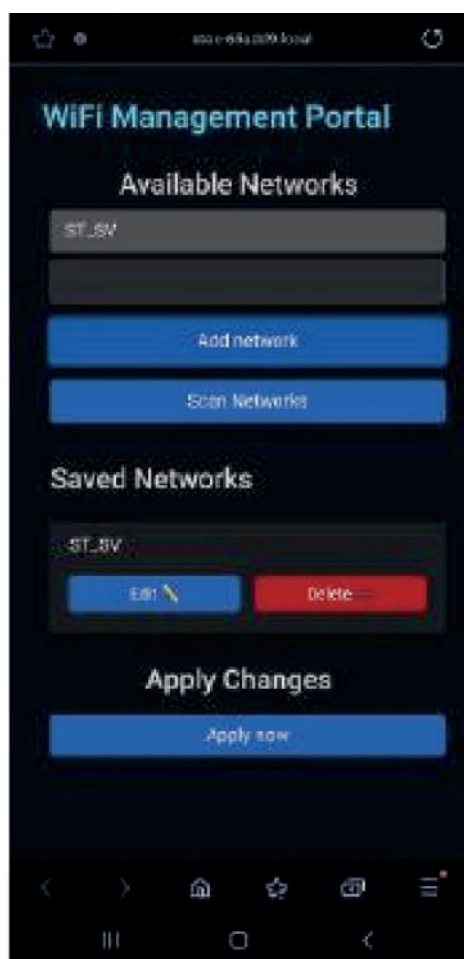
Please be sure that your mobile device and the Sensing Hub are connected to the same Wi Fi network to proceed with the next steps

Scan the QR code (Portal IP) to Access the Web Portal

The QR shown is an example. Use the QR you have received with your Sensing Hub

From the Web Portal you are able to:

- Edit the data of the saved Wireless network
- Delete the saved Wireless Network
- Add a new Wireless Network

If you delete all the saved WiFi networks, you will need to follow the steps from section 9.2 to save a new Wi Fi Network.

10. ACCESS TO THE CLOUD APPLICATION

10.1 You can Access to the Cloud App using any device with Internet connection.
Open your browser and go to the website: <https://www.podisensego.net/>


10.2 Access the registration area to get your credentials
<https://sensingtex.com/registration/>

10.3 Access the Cloud with your credentials
Introduce your credentials (provided by Sensing Tex) to Access to your space

A screenshot of the Podisense GO login interface. At the top, the 'PODISENSE GO' logo is displayed in bold black text. Below the logo, there are two input fields: 'Correo electrónico' (Email) with the placeholder text 'ejemplo@sensinghealth.com' and 'Contraseña' (Password) with a masked password 'XXXXXXXXXX' and an eye icon to toggle visibility. At the bottom of the form is a black button with the white text 'Iniciar sesión' (Log in).

**Need assistance? Email us at
info@podisensego.net**

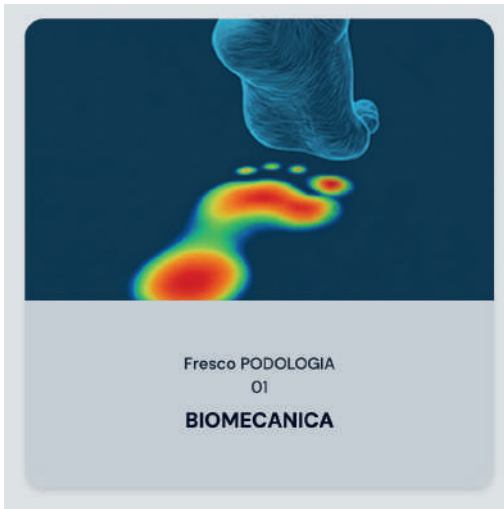11. MAIN SCREEN OF THE MEMBER AREA

Select the device you want to use.

You Will see the screen to operate with the device

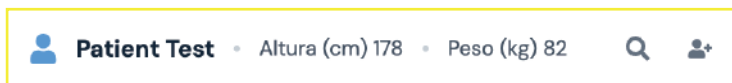
Grey color:
Device offline, not connected

Green color:
Device connected and online

12. DEVICE

Add new patient or edit current user data.

Find an user in the database to select it.

Shows status device and data company

When you select “New Patient”, a form Will be opened to fill the data.

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Cerrar **Guardar**

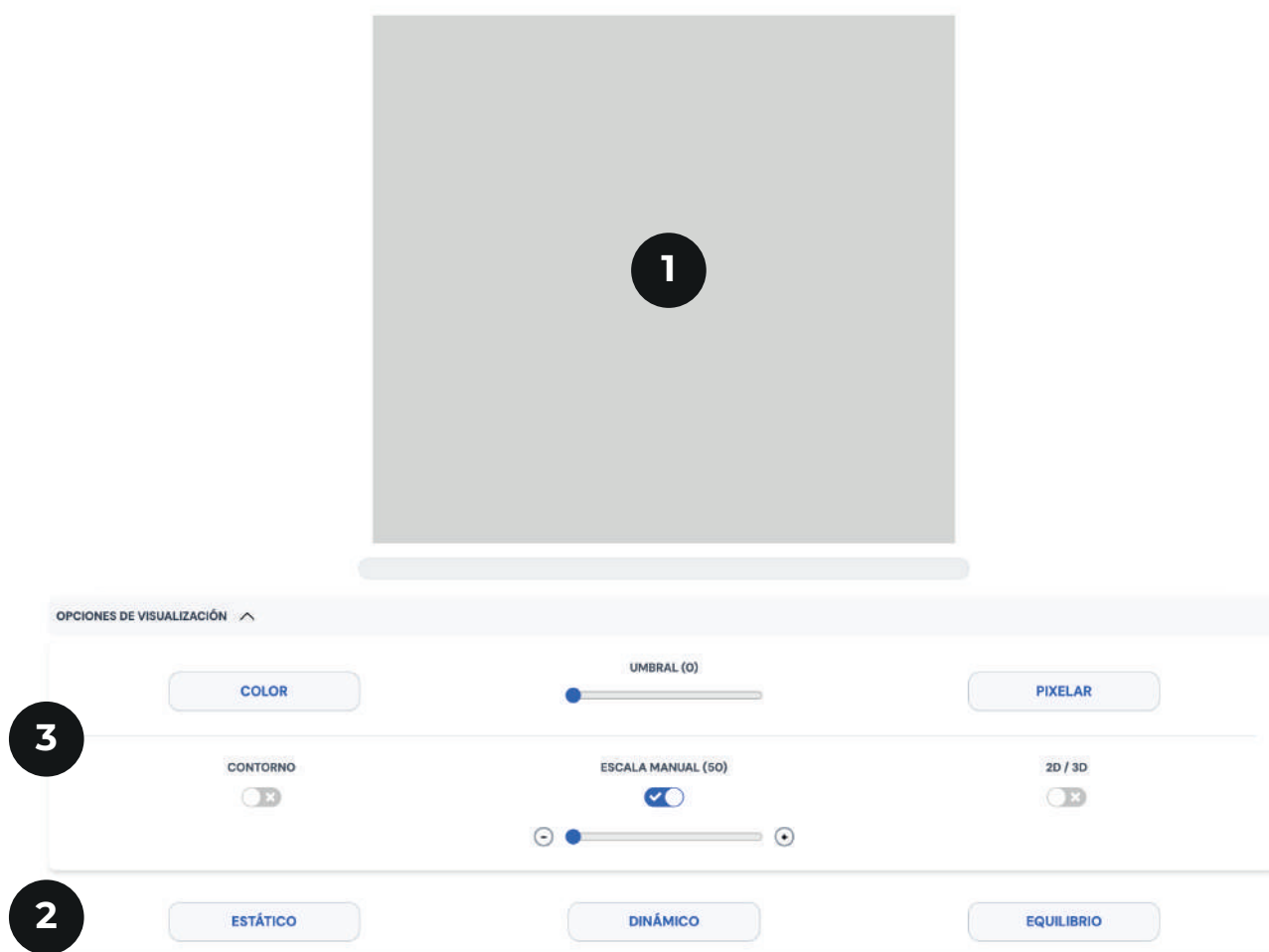
Clicking on  will open a window with the list of patients to select the patient and perform the analysis with the selected patient:

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▼

Cerrar

13. REAL TIME PRESSURE MAPS AND DATA ANALYSIS

1: Real time pressure map: In this Area the pressure map is shown

2: Analysis area: There are three different analysis that could be selected

STATIC ANALYSIS (See section 14.1)

It shows the pressure distribution and some magnitudes of the pressure distribution, maximum pressure and center of pressure

DYNAMIC ANALYSIS (See section 14.2)

Allows to analyze the progress of the steps of each foot and compare them

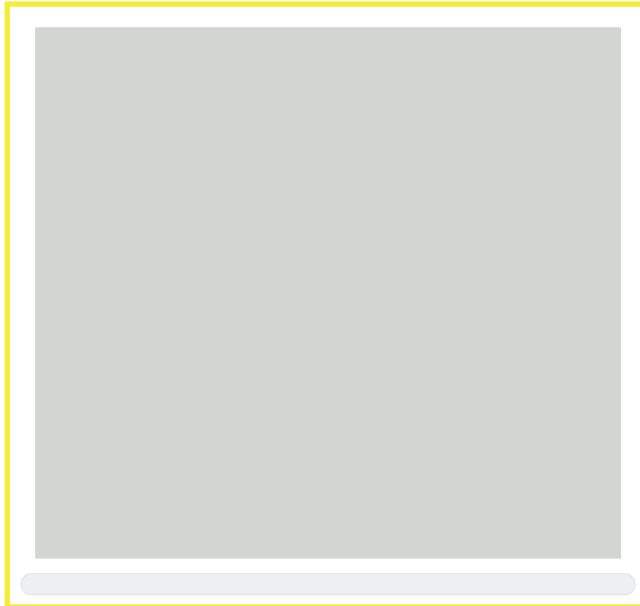
BALANCE ANALYSIS (See section 14.3)

Allows to see the balance of the user over time


3: Display Options (See section 14)

Allows to select different preferences related to the color scale; smooth; contour; 2D/3D view.



14. SOFTWARE VISUALIZATION

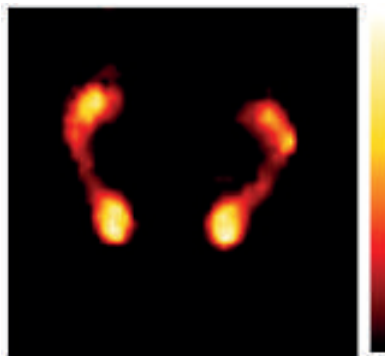
It shows the current pressure mapping applied over the sensor. It is possible to deploy the advanced options in order to find the optimal configuration.


COLOR

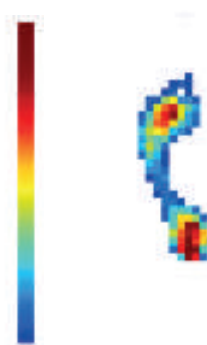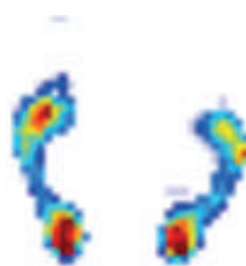
PIXELATE

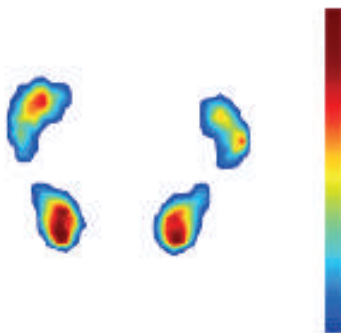THRESHOLD (35)

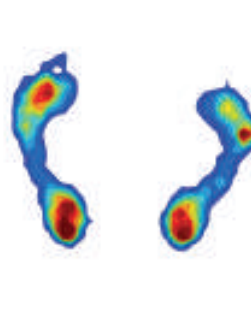CONTOUR

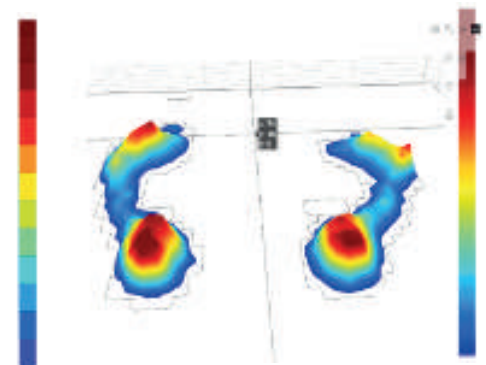2D / 3D

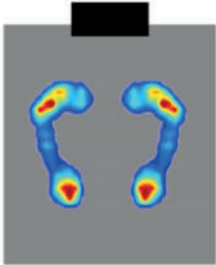
14.1 STATIC ANALYSIS

Press the icon “STATIC ANALYSIS”

A brief tutorial appears after pressing the button. The analysis starts when the READY button is pressed. Follow the instructions provided

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


Listo

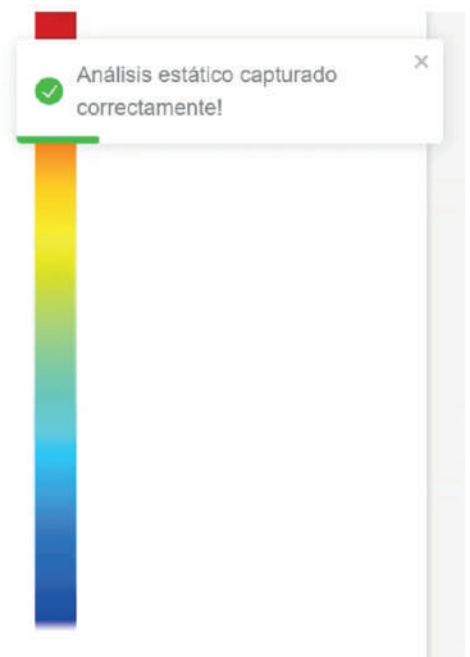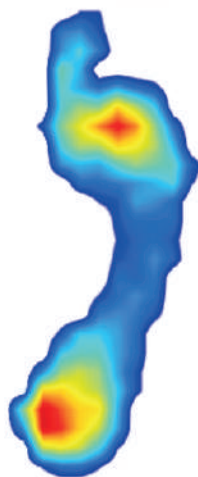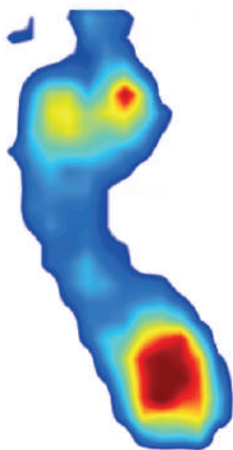To do the analysis you have to visualize the pressure map in real time for safety reasons

Preview of the saved session: (Opening the analysis from the data base)

○ **LEFT / RIGHT FRONT:** Data from the front part of the left/right foot

○ **LEFT / RIGHT GLOBAL:** Data from the entire left/right foot

- Center of Pressure (CoP) Coordinates (X, Y)
- Maximum Pressure Point: Coordinates (X, Y) / Pressure in kPa (Z)

○ **LEFT / RIGHT BACK:** Data from the back part of the left/right foot

- Distribution Percentage of total body weight
- Surface Contact area

○ **GLOBAL:** Data from both feet combined

- Center of Pressure (CoP) Coordinates (X, Y)
- Global Maximum Pressure Points: Coordinates (X, Y) / Pressure in kPa (Z)

Saved reports will appear scrolling down to the ANALYSIS HISTORIC

ANALYSIS HISTORIC			
☐ Only Display User Analysis			
Date	User	Analysis Type	Notes
 12-10-2025, 16:07:11	Miguel Rivas	State	1000

You can edit, download or trash your reports.

Descargar Análisis ×

1

Seleccionar Widgets

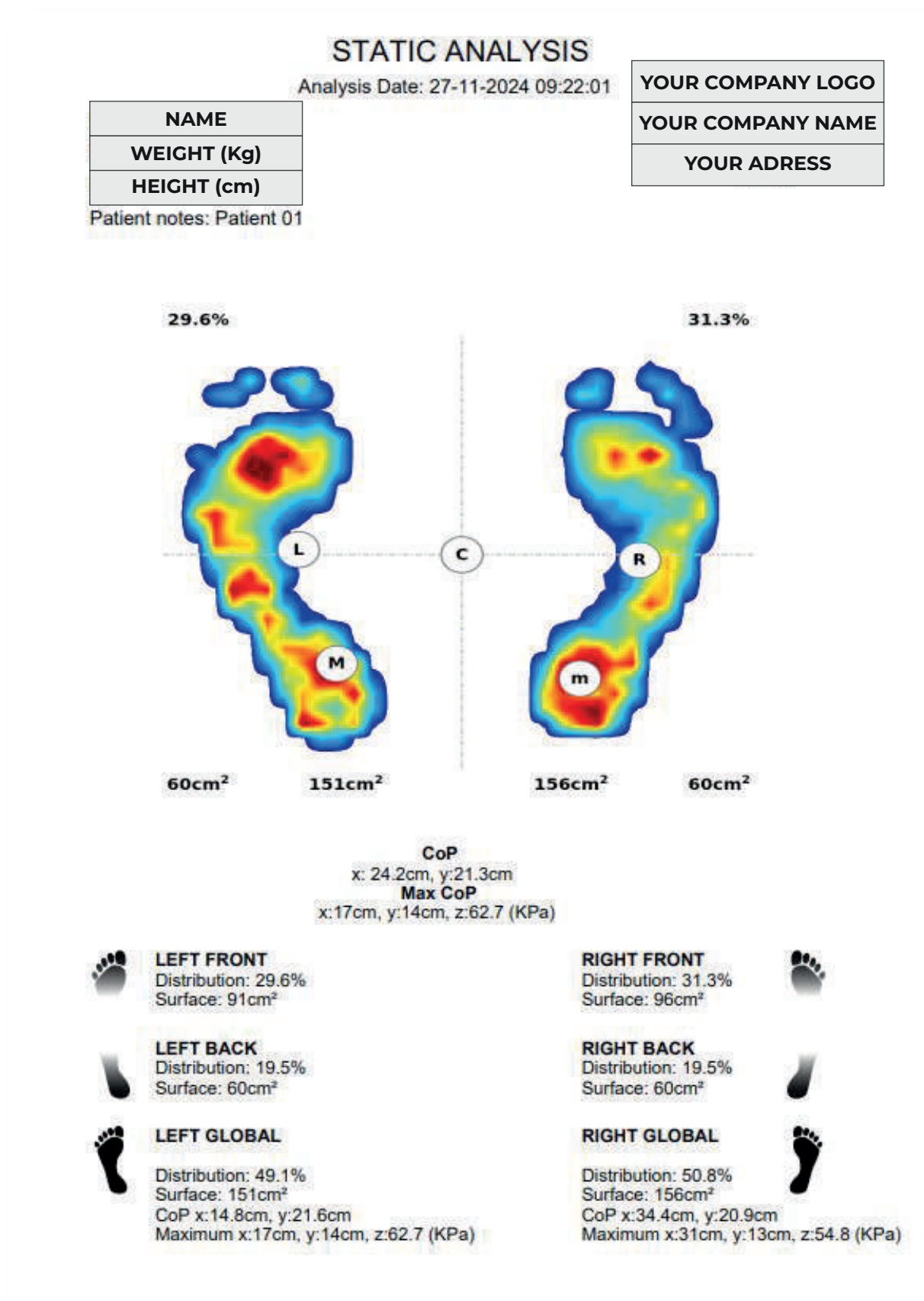
☒ Página principal

☒ Notas

Descargar

When you download, you can select whether or not to print the notes.

PDF format document with the analysis:


14.2 DYNAMIC ANALYSIS

Press the icon “DYNAMIC ANALYSIS”

A brief tutorial appears after pressing the button. The analysis starts when the READY button is pressed. Please follow the instructions to execute the analysis

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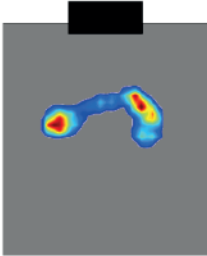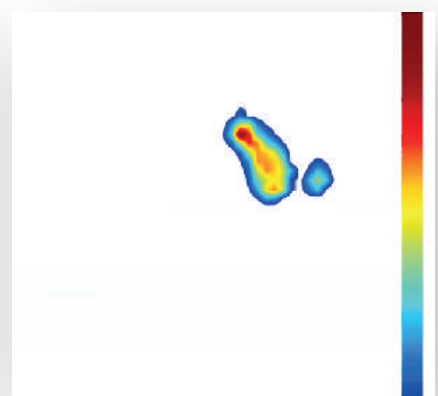
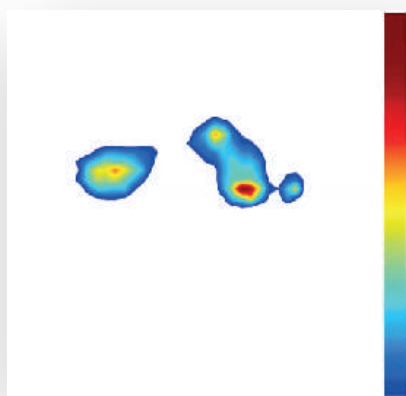
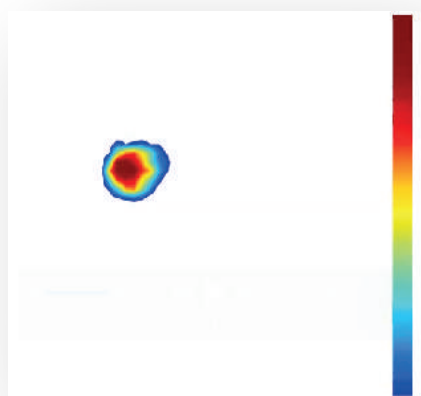
Listo

A timer will appear. The patient should begin walking when the capture cycle counter starts. The patient has to start from the left side of the sensor, placing their left foot first, and then follow the same sequence for the right foot from the right side


To do the analysis you have to visualize the pressure map in real time for safety reasons

Saved reports will appear scrolling down to the ANALYSIS HISTORIC

ANALYSIS HISTORIC				
☐ Only Display User Analysis				
Date	User	Analysis Type	Notes	
17-10-2025, 16:07:11	Miguel Ruiz	Static	1000	  

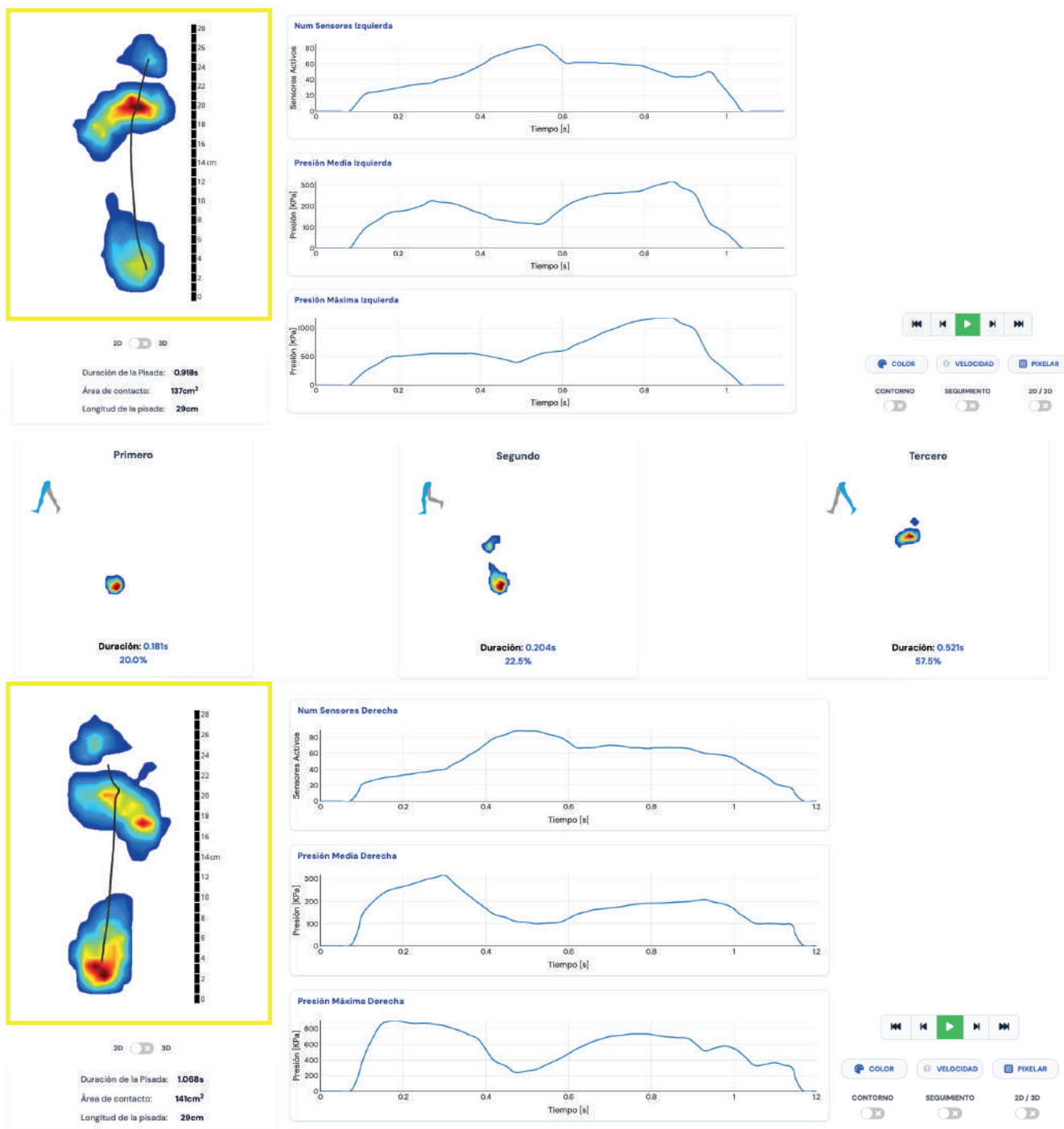Preview of the saved session: (Opening the analysis from the data base)

BLACK LINE: Evolution of the CoP during the step

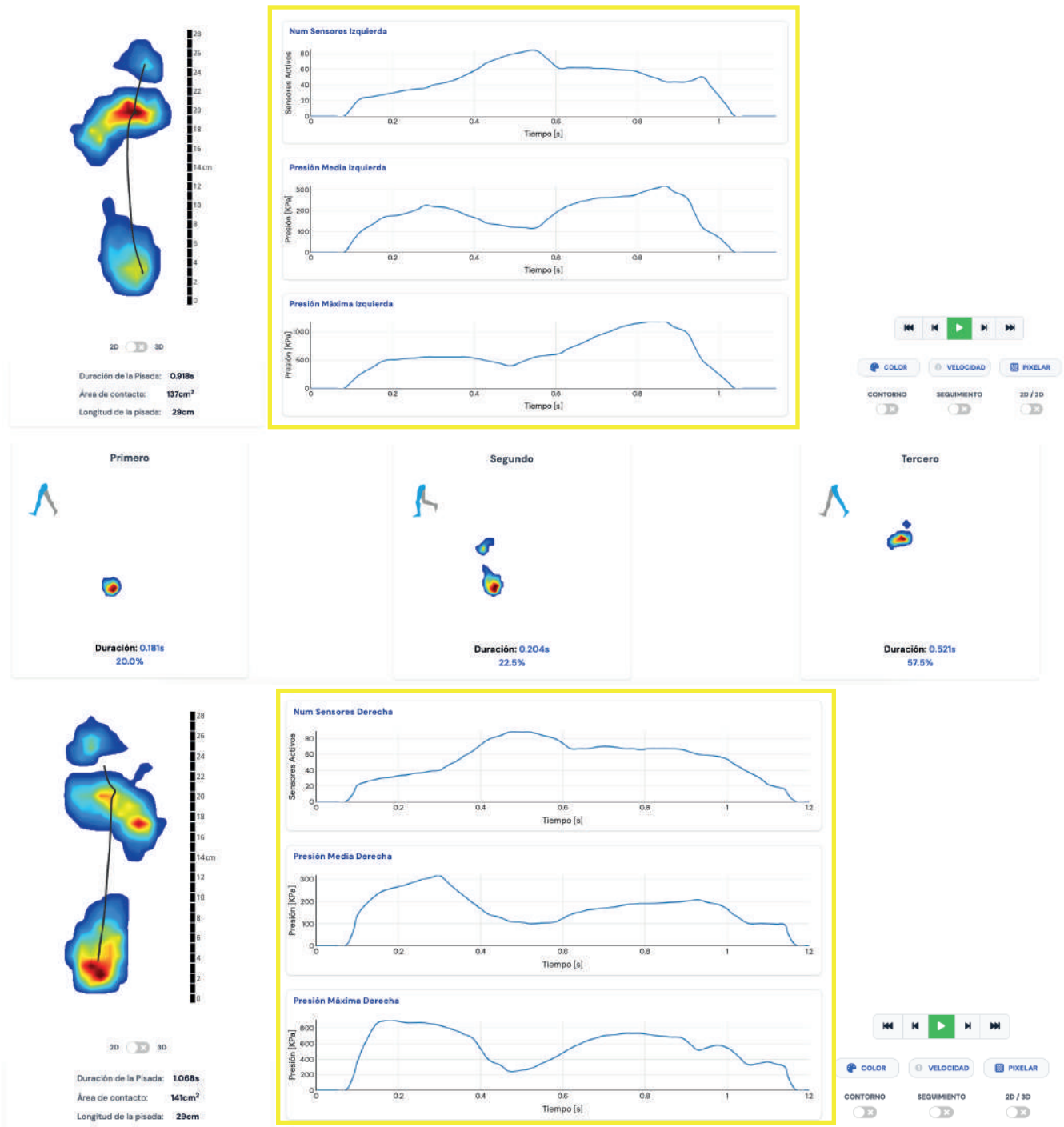
STEP TIME: Duration it takes to complete a full step (heel strike to toe off)

CONTACT AREA: Total surface area of the foot that comes into contact with the ground. (cm²)

STEP HEIGHT: Size of the step vs horizontal of the platform (cm)

Note: this is not the foot height or the foot size

Preview of the saved session: (Opening the analysis from the data base)


NUMBER OF SENSORS LEFT / RIGHT: Number of pressure sensors activated on each foot

AVERAGE PRESSURE LEFT / RIGHT: Average pressure detected on each foot during ground contact, represented in a graph. Pressure values are shown in kilopascals (kPa) and seconds.

MAX PRESSURE LEFT / RIGHT: Highest pressure recorded on each foot during ground contact, represented in a graph. Pressure values are shown in kPa and time in seconds.

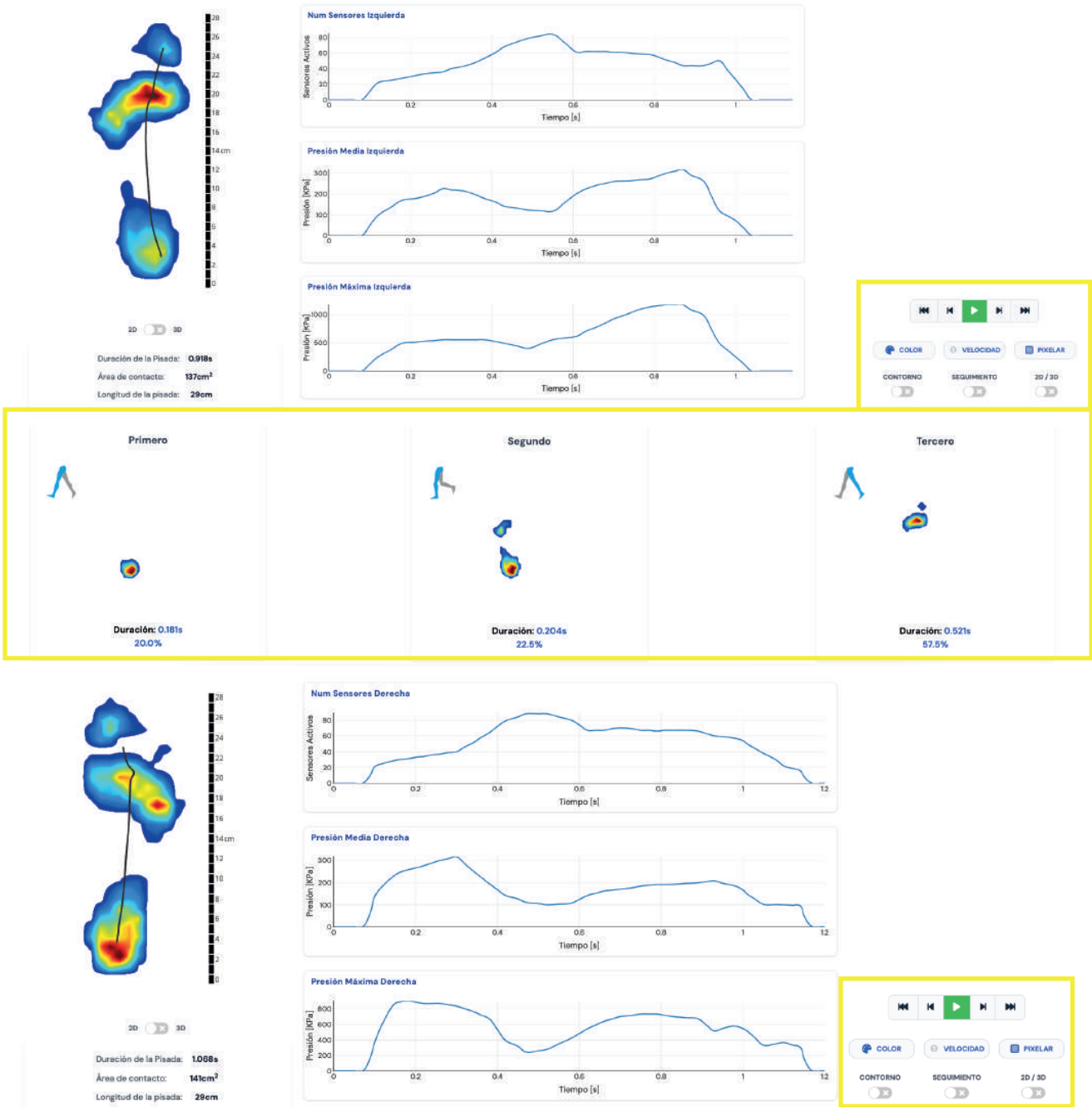It is possible to see a video of each foot with all the sequence of frames captured. The same advanced options are available to optimize the representation:

GAIT ROCKERS Duration (seconds) & percentage of the total step time

LEFT / RIGHT HEEL ROCKER: Phase when the heel first contacts the ground

LEFT / RIGHT ANKLE ROCKER: Phase when the foot is flat and the tibia rotates forward over the ankle.

LEFT / RIGHT FOREFOOT ROCKER: Phase when weight shifts to the forefoot and the heel lifts off the ground.

Saved reports will appear scrolling down to the ANALYSIS HISTORIC

ANALYSIS HISTORIC				
Only Display User Analysis				List Details
Date	User	Analysis Type	Notes	
 17-10-2025, 18:07:11	Miguel Rivas	State	100	  

You can edit, download or trash your reports.

Download Analysis

1 Select Widgets

Main Page

Rockers

Notes

No notes available

Download

2 Download Real Scale

Select Size

A3

A4

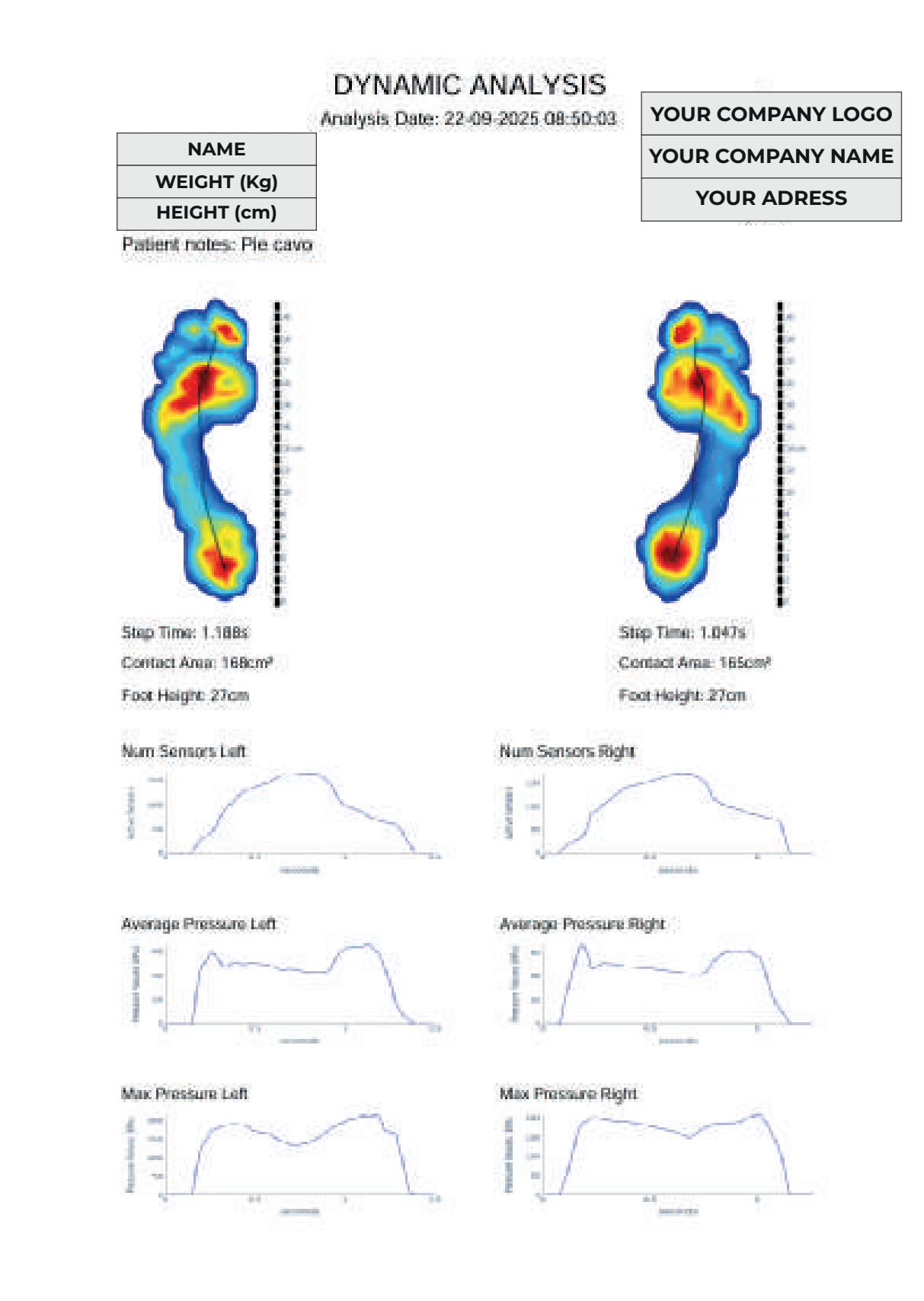
Info

Download


When you download, you can select whether or not to print some widgets.

You can also select to print the pressure map on different scales.

PDF format document with the analysis:


PDF format document with the analysis:

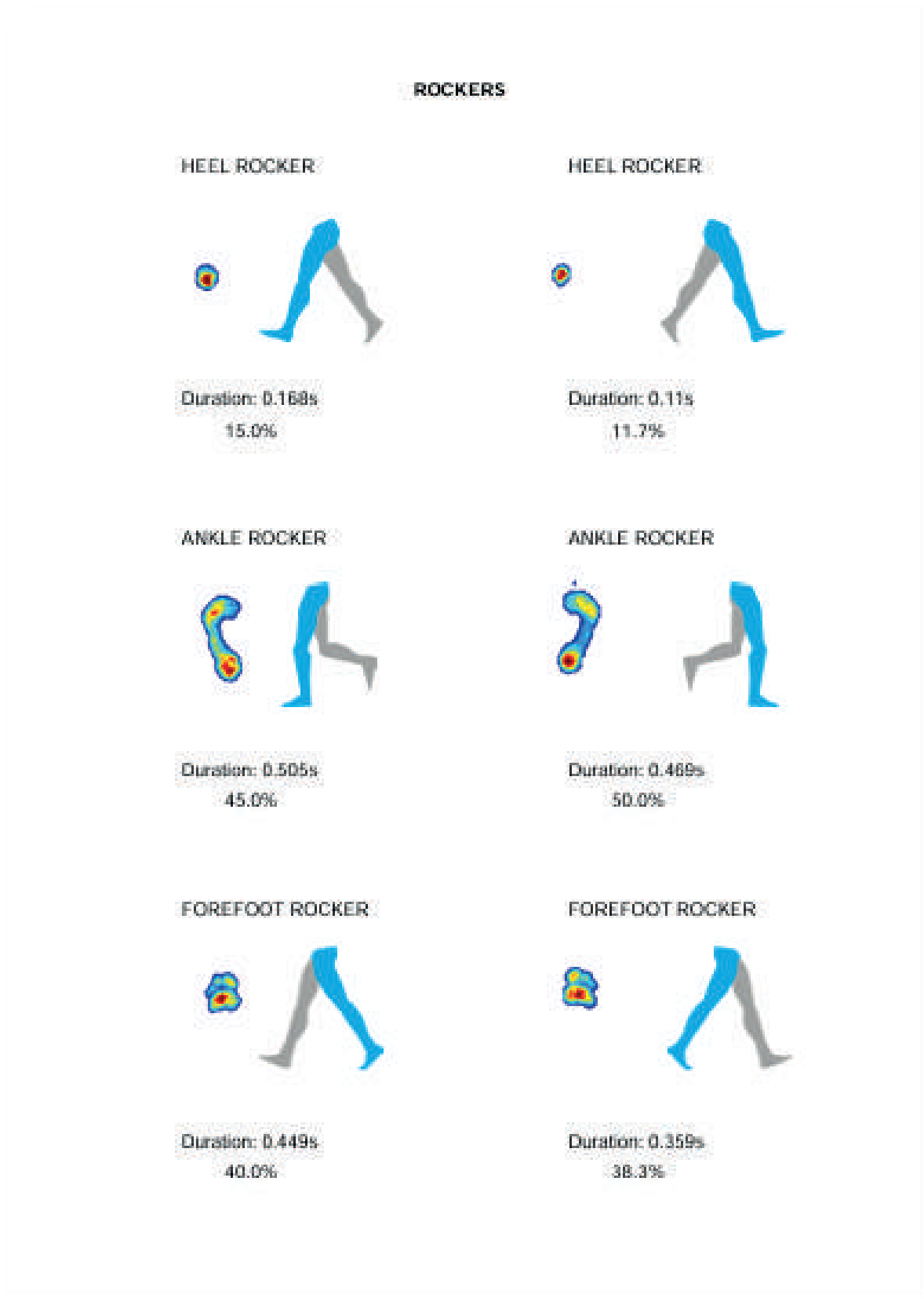14.3 EQUILIBRIUM ANALYSIS

It analyzes the behavior of the balance of the user throw a lapse of time:

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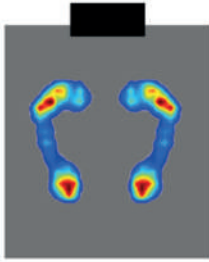Listo

Press the icon “BALANCE ANALYSIS” the information. A tutorial will appear, please follow the instructions of the tutorial to execute the analysis. Press Ready to start

To do the analysis you have to visualize the pressure map in real time for safety reasons

It analyzes the behavior of the balance of the user throw a lapse of time. Also, it is available the analysis of the lateral, front and back balance analyzed.

You can select different visualizations

Saved reports will appear scrolling down to the ANALYSIS HISTORIC

ANALYSIS HISTORIC			
☐ Only Display User Analysis			
Date	User	Analysis Type	Notes
 17-10-2025, 16:07:11	Miguel Rufin	State	Y00

You can edit, download or trash your reports.

Descargar Análisis ×

1

Seleccionar Widgets

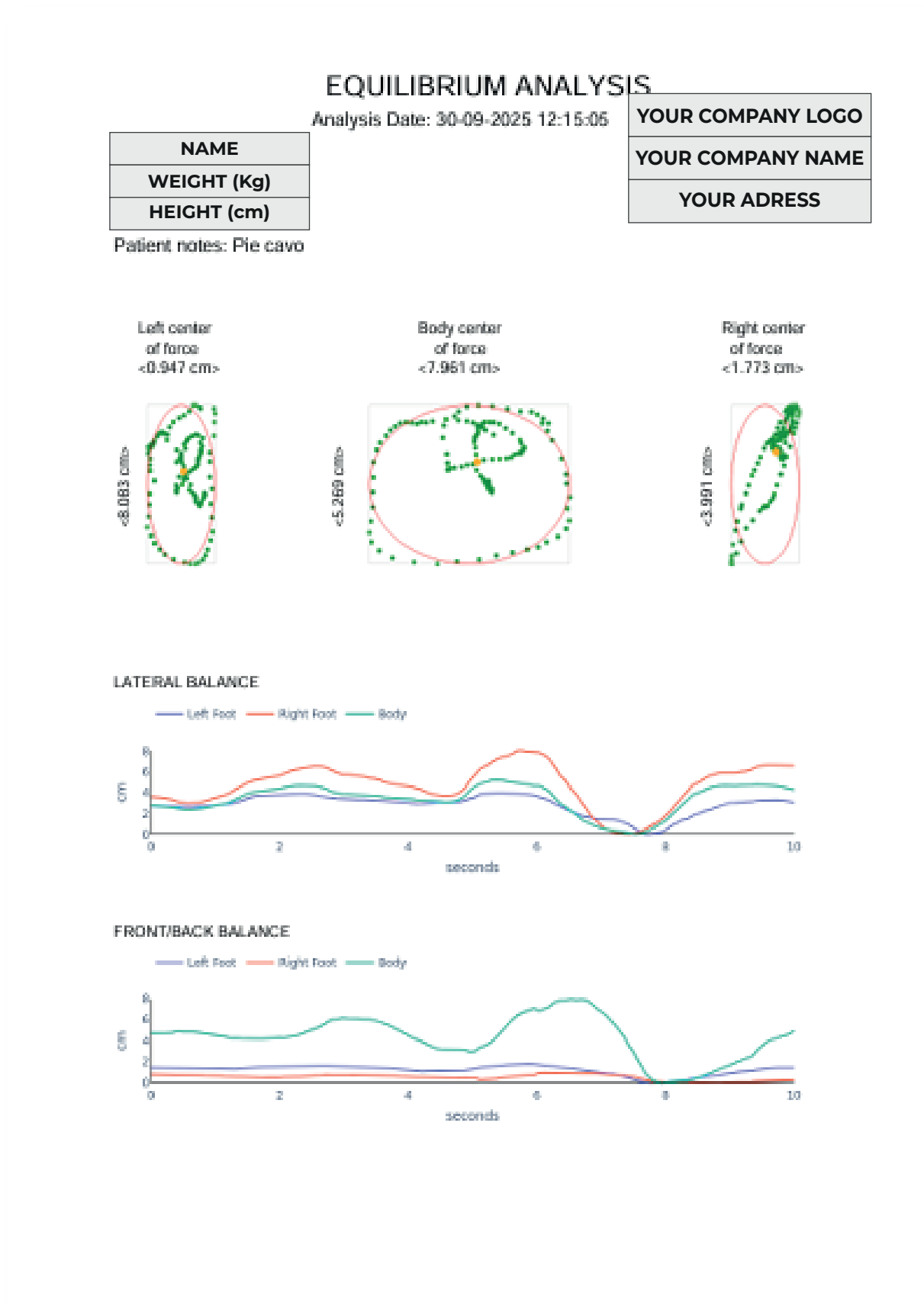
☒ Página principal

☒ Notas

Descargar

When you download, you can select whether or not to print the notes.

PDF format document with the analysis:


15. DATA BASE

The results of the analysis are stored on a database.

The bar indicates the space available in the database to store analysis:

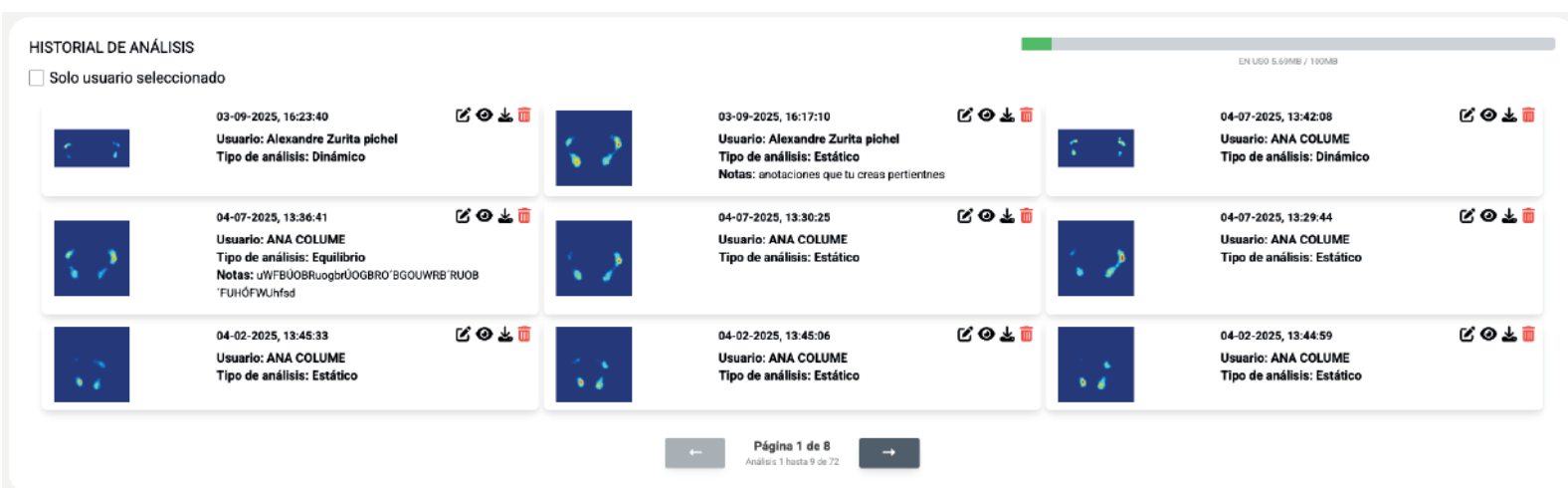The stored analysis are organized in pages. Each page shows 9 analysis that could be edited, opened, downloaded or deleted using the icons as follows.

A screenshot of the 'HISTORIAL DE ANÁLISIS' (Analysis History) interface. At the top, there's a title 'HISTORIAL DE ANÁLISIS' and a checkbox 'Solo usuario seleccionado'. Below this is a grid of 9 analysis cards arranged in 3 rows and 3 columns. Each card contains a small thumbnail image, a timestamp, a username, an analysis type, and a list of notes. To the right of each card are four icons: a pencil (edit), an eye (preview), a download arrow (download), and a trash can (delete). At the bottom of the grid, there's a pagination bar showing 'Página 1 de 8' and 'Análisis 1 hasta 9 de 72'. A progress bar at the top right indicates 'EN USO 5.69MB / 100MB'.

Allows to add or edit notes to the analysis

Allows to open the analysis to do a preview

Allows to download the analysis as a PDF file

Allows to delete the register

15.1 ANALYSIS HISTORIC

The results of the analysis are stored on a database

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The bar indicates the space available for you account in the database to store analysis.

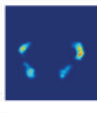
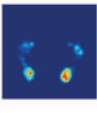
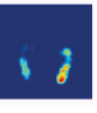
The stored analysis are organized in pages. Each page shows 12 analysis that could be edited, opened, downloaded or deleted using the icons as follows

The analysis can be visualized either in a list view or in a card view

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16. MAINTENANCE GUIDE OF THE PLATAFORM

Layer F07 Neoprene is an opaque, smooth feel and spongy substrate for Mats. The 3 D structure allows to use this fabric in soft environments to increase the comfort.

PROPERTY	VALUE
Color	Black
Grammage [g/ m2]	400
Thickness [mm]	3
Composition	96% PES 4% SPANDEX

Sterilization process using alternative products without bleach

- **Soap and Water** For general cleaning and sterilization, you can wash neoprene with mild soap and warm water Use a soft cloth or sponge to gently clean the neoprene surface Rinse thoroughly with clean water to remove any soap residue and then allow it to air dry
- **Disinfectant Wipes** You can use disinfectant wipes or a cloth soaked in a mild disinfectant solution to wipe down the neoprene surface
- **Isopropyl Alcohol** Isopropyl alcohol can be used to disinfect neoprene Dampen a cloth with isopropyl alcohol and wipe down the neoprene surface Ensure the neoprene is completely dry before using it again
- **UV C Light** Some neoprene items can be sterilized using UV C light, which kills bacteria and viruses
- Do not bleach, do not tumble dry, do not iron, do not dry clean

17. DECLARATION OF CONFORMITY CE MARK

SENSING TEX, S.L.
Avinguda Cerdanyola, 75
08172 Sant Cugat del Valles,
Barcelona, Spain

CE CERTIFICATE / EU DECLARATION OF CONFORMITY

This declaration of compliance is issued under the exclusive responsibility of the manufacturer or Legal Representative authorized by the EEE:

Description and identification of the product to which this declaration refers:

- Name of the Product: SENSING HEALTH PODIATRY
- Model: 1.0
- Serial Number: [stex-#####.com](#)

The indicated product complies with the Community Harmonization legislation concerned:

- ETSI EN 301 489-1 V2.2.3 (2019)
- UNE-EN 61000-6-3:2021
- UNE-EN 61000-6-1

The tests were carried out by the DIGIPROCES Laboratories

All the particular conditions applicable to the usage of the product are indicated in the Instructions Manual

Signed on behalf:
Miguel Ridao Granado
Sensing Tex, SL

Sensing Tex
CIF: B-66264293

Name: Miguel Ridao Granado
Position: General Manager
Barcelona, on the 30th of September 2023

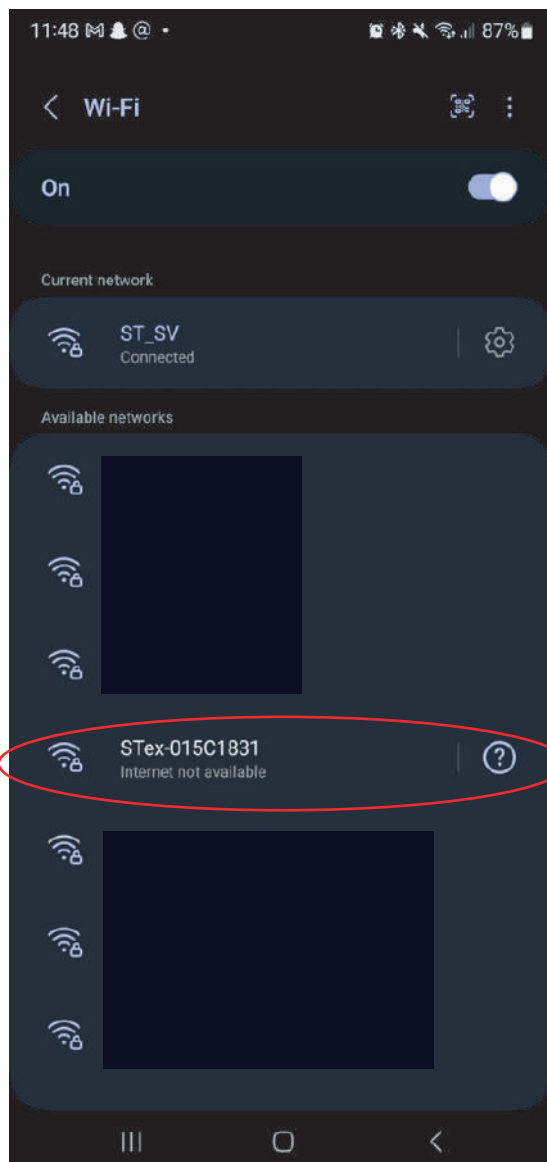
18. FREQUENT ANSWERED QUESTIONS

1. IS IT POSSIBLE TO DO THE WIFI SETUP MANUALLY?

Yes this is possible, If you are having troubles using the QR codes provided, you can set up the Wi Fi configuration manually following the next steps:

a. Set up the Wi Fi configuration for the first time


Open the WiFi manager in your mobile phone and select the Wi Fi that corresponds with the label of your Podisense Hub device

STex-5C1831

The password is the same ID shown in the Wi Fi settings 015C1831 in this example

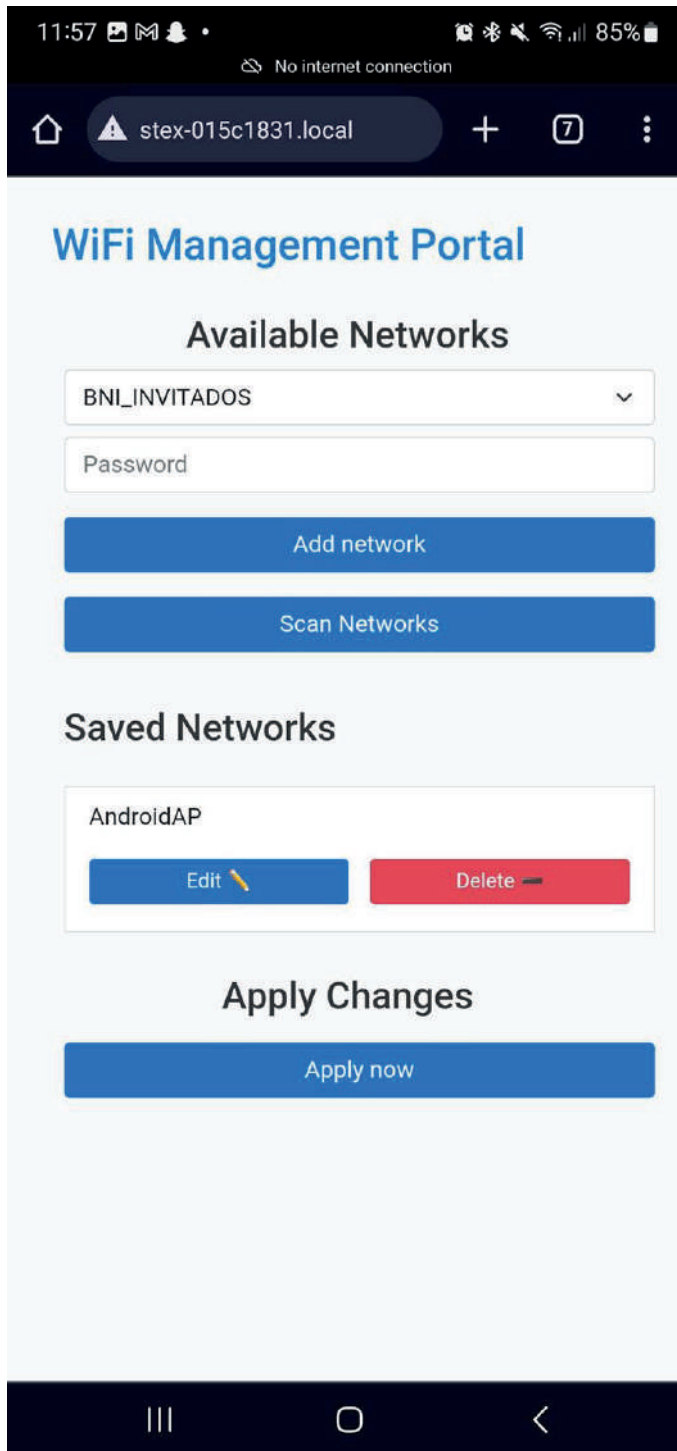1. IS IT POSSIBLE TO DO THE WIFI SETUP MANUALLY?

Open a browser in your device (Chrome i.e.) and introduce the URL using the provided ID.

stex-015c1831.local (in this example)

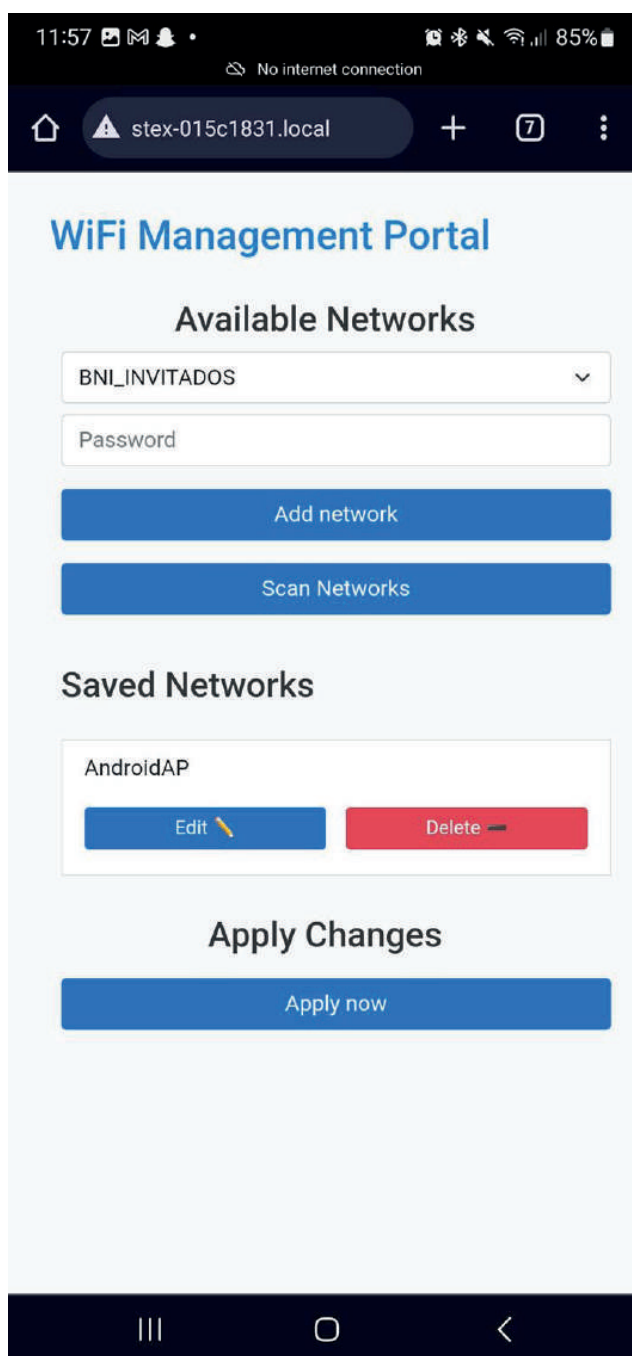
You Will get Access to the WiFi Management portal to configure the new Wi Fi network

1. IS IT POSSIBLE TO DO THE WIFI SETUP MANUALLY?

b. Set up a new Wi Fi when you are connected to a Wi Fi network; or manage the saved networks

Please, be sure your mobile device is connected to the same network as the Podisense Hub

Open a browser in your device (Chrome i.e.) and introduce the URL using the provided ID.

stex-015c1831.local (in this example)

You Will get Access to the WiFi Management portal to configure the new Wi Fi network

You are in the equivalent point as section 9.1 of this document

2. WHAT HAPPENS IF THE IMAGE IN THE CLOUD APP HAVE A DELAY?

If you are experiencing delays in the reaction time while using the Sensing Health Podiatry; could be due because your device is not powerful enough to upgrade the images in real time.

Please double check that you have enough bandwidth in your internet connection.

3. WHAT HAPPENS IF WE DO NOT SEE ANY IMAGE STREAMED ON THE APP?

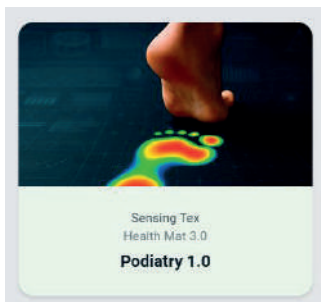
1. Be sure that the Podisense Hub is correctly connected and powered The red LED should be ON

2. Be sure that the Sensor is correctly connected and powered The green LED should be ON

3. Be sure that you are logged with the Web App with the correct user and password

4. Be sure that the device appears in the Cloud App and it appear in green color

5. Double check that you are not applying security rules that are blocking the Podisense Hub internet connection

QUICK START GUIDE

1 Plataforma Podisense GO

2 Carrying bag packaging materials

3 Podisense "HUB"

4 Power cable

5 LAN cable
Micro USB cable

INSTALL THE PLATFORM

- Open the carrying bag, remove the platform, take off the Velcro strap, unroll it, and place the platform on a flat, hard surface.
-  Connect the micro-USB to the platform, paying attention to the polarity.
- Plug the other end of the USB cable into either of the two available USB ports on the Podisense HUB.
- Connect the power cable to the Podisense HUB (USB-C) and plug it into the wall outlet.
- Make sure power is reaching the cable by checking that the green light on the Podisense GO platform is on.

INTERNET CONNECTION AND SOFTWARE SETUP

This platform must be connected to the internet to operate, and the Podisense Hub makes that possible. Podisense Hub is a next-generation microprocessor—ultralight, compact, and extremely low-power—allowing you to use the platform anywhere, even with a power bank.

Below, you'll find step-by-step instructions to set up your Podisense Hub. Inside the carrying case, you will find 1 sheet with 6 QR codes.

OPTION 1: Ethernet (LAN) connection The simplest and most stable method.

- 1- Connect the LAN cable to the Podisense Hub.
- 2- Scan **QR #5**. A webpage will open where you must fill in your details. You will receive an email with your login credentials.
- 3- Scan **QR #4** to access the Client Area at **podisensego.net** → Client Area.

OPTION 2: WiFi connection

For this option, you will need a mobile phone.

Before you start: *Disable mobile data on your phone (very important).*

WiFi Setup

- 1- Scan **QR #1**.
 - Your phone will warn you that the network has no internet.
 - Select "Connect only this time."
- 2- Scan **QR #2** (Portal – host name). The **WiFi Management** page will open.
- 3- Find your WiFi network, select it, and enter the password.
- 4- Tap **Add Network** → a confirmation message will appear.
- 5- Tap **Apply Now** → you will see another message (Rebooting).
- 6- Wait 1–3 minutes. After rebooting, your Podisense Hub will be connected to that WiFi network.

Note: The Hub can be configured with as many WiFi networks as you need, including a hotspot from another mobile phone. Please keep in mind that networks with firewalls may require technical assistance

- 7- Scan **QR #5** to obtain your login credentials.
- 8- Scan **QR #4** to access the Client Area at **podisensego.net** → Client Area.

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